

Beltone HearPlus™ app for Apple Watch™

Control your hearing aid at all times.

How to get the Beltone HearPlus app on your Apple Watch

What you need

The Beltone HearPlus app must be installed on your iPhone and updated to minimum v. 3.0.1. Please see Apple compatibility information on p. 3.

How to pair

Pair your Apple Watch to your iPhone by following the instructions found in the companion app for Apple Watch on your iPhone.



How to get the app on Apple Watch

You can use the companion app for Apple Watch on your iPhone to turn on or off or to set preferences for your apps on Apple Watch, including the Beltone HearPlus app.

TIP HOW DO I MOVE BETWEEN SCREENS?

Simply slide sideways to move between screens.

VOLUME CONTROL



Hearing aid volume control

Tap the + or – icons to increase or decrease the volume of the hearing aid.

To mute the hearing aids, force touch on the screen. To unmute, tap the + or – icons.



Streamer volume control

When a streamer program is selected you can adjust the streaming volume separately with the orange volume slider.

You can also mute the streaming volume by Force Touch.

PROGRAMS AND FAVORITES



Choose a program or a Favorite

Tap one of the program icons to choose a program. Scroll down to choose between hearing aid programs, streamer programs and your own Favorites.

You can create new Favorites in the Beltone HearPlus app on the iPhone.

BASS AND TREBLE



Adjust bass and treble

Tap the + or – icons to increase or decrease bass and treble.

SOUND ENHANCER

Adjust Sound Enhancer

Adjust all three features by tapping the + and – icons. Use force touch to reset to fitted settings.



Comfort in Noise

Adjust the level of comfort in noise by turning noise reduction up or down.



Speech Focus

Adjust speech focus by setting beam width.



Comfort in Wind

Adjust comfort in wind by turning wind noise reduction up or down.

TIP CAN I USE APPLE WATCH GLANCES FOR THE BELTONE HEARPLUS APP?

You can use Glances to quickly view which program you're in. Just swipe up on the Watch face and then swipe sideways until you see the Beltone HearPlus app. It will tell you your current program. Tap the screen to go directly to the app.
